Ambu® Neuroline Concentric Neurology Needle Electrode

Instructions for use

Instructions for use

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Kullanma Talimatı

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Instructions for use

Single use Ambu® Neuroline Concentric Neurology Needle Electrode

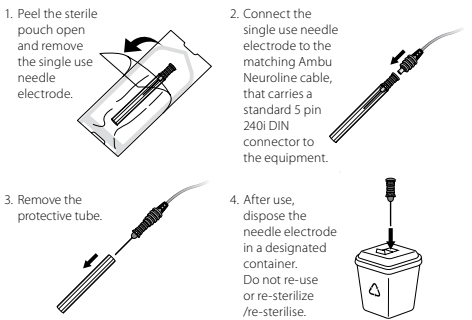
Intended Use
The Ambu® Neuroline Concentric Needle Electrode are made for muscle activity recording for Electromyography (EMG) applications. The needle electrodes are for single use only.

Precautions
<ol style="list-style-type: none">The Ambu® Neuroline Concentric Needle Electrode shall be used by health care professionals trained in performing neurophysiological examination. For single use only. Reuse of product can cause contamination, leading to infections. Do not soak, rinse or sterilize this device as these procedures may leave harmful residues or cause malfunction of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures. Electrodes must only be used with Electromyography (EMG) applications monitoring devices in compliance with IEC 60601-1. During use, the electrodes should be placed as far as possible from the equipment which generates electromagnetic interference (EMI) to minimize unwanted radiofrequency interference (RFI). Do not use if individual pouch is damaged or has been opened. Rx only.

Sterilization/Sterilisation
Sterilized/ Sterilised by Electron Beam Irradiation (R-sterilized/ R-sterilised).

Packaging
One needle electrode in a pouch.

Directions for use

Ambu® is a registered trademark of Ambu A/S, Denmark. Ambu A/S is certified according to ISO 13485

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Ambu® es una marca registrada de Ambu A/S, Dinamarca
Ambu A/S dispone de certificado según la norma ISO 13485
Ambu® e registrirana ochranná známka spoločnosti Ambu A/S, Dánsko
Spoločnosť Ambu A/S je certifikována dle norem ISO 13485
Ambu® er et registreret varemærke og tilhører Ambu A/S, Danmark
Ambu A/S er ISO 13485-certificeret